



EXCLUDER®

AAA ENDOPROSTHESIS

Sizing Guide

Trunk-Ipsilateral Leg Endoprosthesis Sizing Guide:¹

Intended Aortic Vessel Diameter (mm)	Aortic Endoprosthesis Diameter ^{2,3} (mm)	Intended Iliac Vessel Diameter (mm)	Iliac Endoprosthesis Diameter ^{2,4} (mm)	Overall Device Lengths (cm)	Recommended Introducer Sheath (Fr x cm)
19 - 21	23	10 - 11 12 - 13,5	12 14,5	12 / 14 16 / 18	18 x 30
22 - 23	26	10 - 11 12 - 13,5	12 14,5	12 / 14 16 / 18	18 x 30
24 - 26	28,5	10 - 11 12 - 13,5	12 14,5	12 / 14 16 / 18	18 x 30
27 - 28	31	12 - 13,5	14,5	15 / 17	20 x 30

Contralateral Leg Endoprosthesis Sizing Guide:¹

Intended Iliac Vessel Diameter (mm)	Iliac Endoprosthesis Diameter ^{2,4} (mm)	Overall Device Lengths (cm)	Recommended Contralateral Introducer Sheath (Fr x cm)	Recommended Angioplasty Balloon Size (mm x mm)
10 - 11	12	10 / 12 / 14 (with trunk, total length = 14 / 16 / 18 cm)	12 x 30	12 x 40
12 - 13,5	14,5	10 / 12 / 14 (with trunk, total length = 14 / 16 / 18 cm)	12 x 30	14 x 40
13,5 - 14,5*	16	9,5 / 11,5 / 13,5 (with trunk, total length = 13,5 / 15,5 / 17,5 cm)	18 x 30	16 x 40
14,5 - 16,5*	18	9,5 / 11,5 / 13,5 (with trunk, total length = 13,5 / 15,5 / 17,5 cm)	18 x 30	18 x 40
16,5 - 18,5*	20	9,5 / 11,5 / 13,5 (with trunk, total length = 13,5 / 15,5 / 17,5 cm)	18 x 30	20 x 40

Aortic Extender Sizing Guide:¹

Intended Aortic Vessel Diameter (mm)	Aortic Extender Diameter ^{2,3} (mm)	Endoprosthesis Length (cm)	Recommended Introducer Sheath (Fr x cm)
19 - 21	23	3,3	18 x 30
22 - 23	26	3,3	18 x 30
24 - 26	28,5	3,3	18 x 30
27 - 29	32	4,5	20 x 30

Iliac Extender Sizing Guide:¹

Intended Iliac Vessel Diameter (mm)	Iliac Extender Diameter ² (mm)	Endoprosthesis Length ³ (cm)	Recommended Introducer Sheath (Fr x cm)	Recommended Balloon Size (Proximal) (mm)	Recommended Balloon Size (Distal) (mm x mm)
8 - 9	10	7	12 x 30	14	10 x 40
10 - 11	12	7	12 x 30	14	12 x 40
12 - 13,5	14,5	7	12 x 30	14	14 x 40

¹ Note: All dimensions are nominal.


² Recommended endoprosthesis oversizing relative to the aortic vessel is approximately 10 - 21% and 7 - 25% for the iliac vessel.

³ Large diameter, low pressure aortic balloon is recommended for the aortic vessel.

⁴ Recommended angioplasty balloon size is 12 mm and 14 mm respectively.

⁵ 7 cm long Iliac Extender provides a maximum extension of 4 cm when placed in the Trunk-Ipsilateral or Contralateral Leg Endoprosthesis.

* Large Diameter Contralateral Leg can also be used as an Iliac Extender.

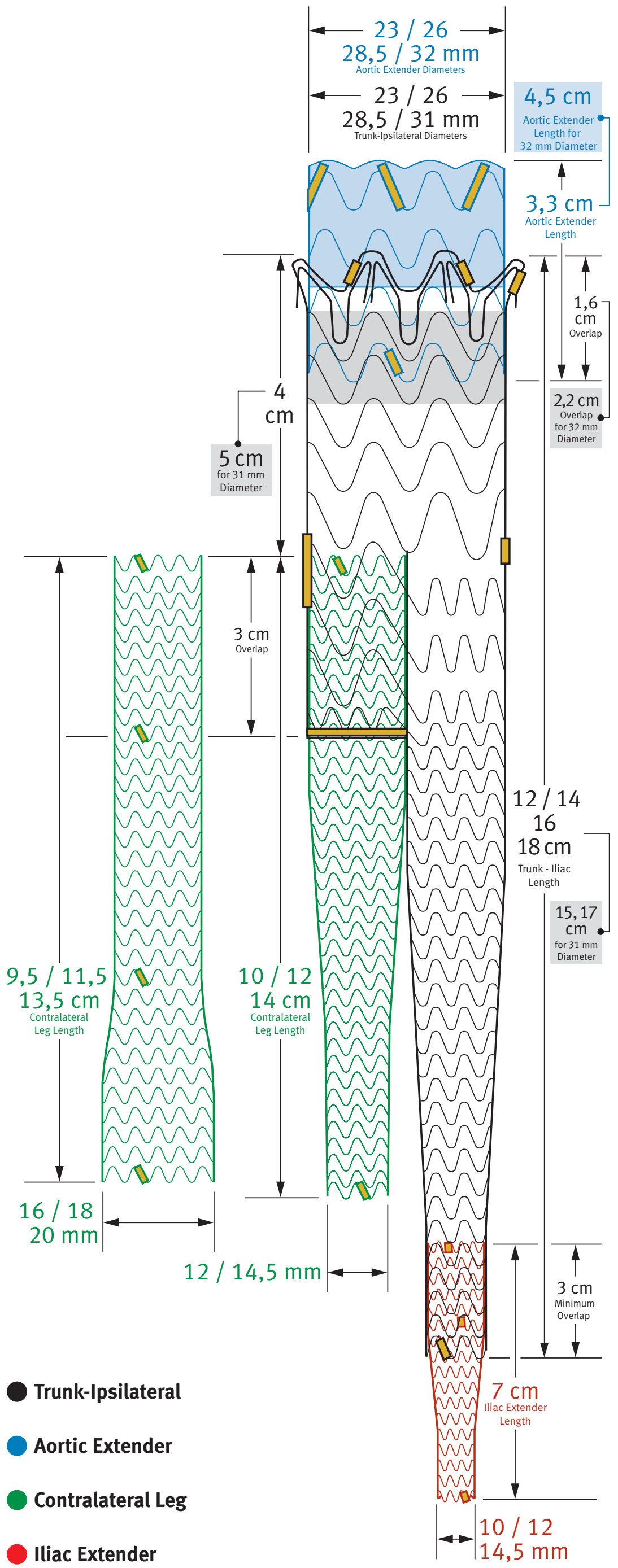

Accessories Checklist*

- 1 - 12 Fr x 30 cm Sheath
- 1 - 18 Fr x 30 cm Sheath or 20 Fr x 30 cm Sheath**
- 2 - 0,035" Superstiff Guidewires
- 1 - Pigtail Catheter
- 1 - Aortic Balloon (appropriate size)
- 1 - 14 mm x 4 cm PTA Balloon
- Miscellaneous Catheters and Wires for Access

* Minimal checklist


** An additional 18 Fr x 30 cm Sheath is required for use of Large Diameter Contralateral Legs

W. L. GORE & ASSOCIATES, INC.

Flagstaff, AZ 86001 • USA

00800.6334.4673

For international contact information, visit goremedical.com

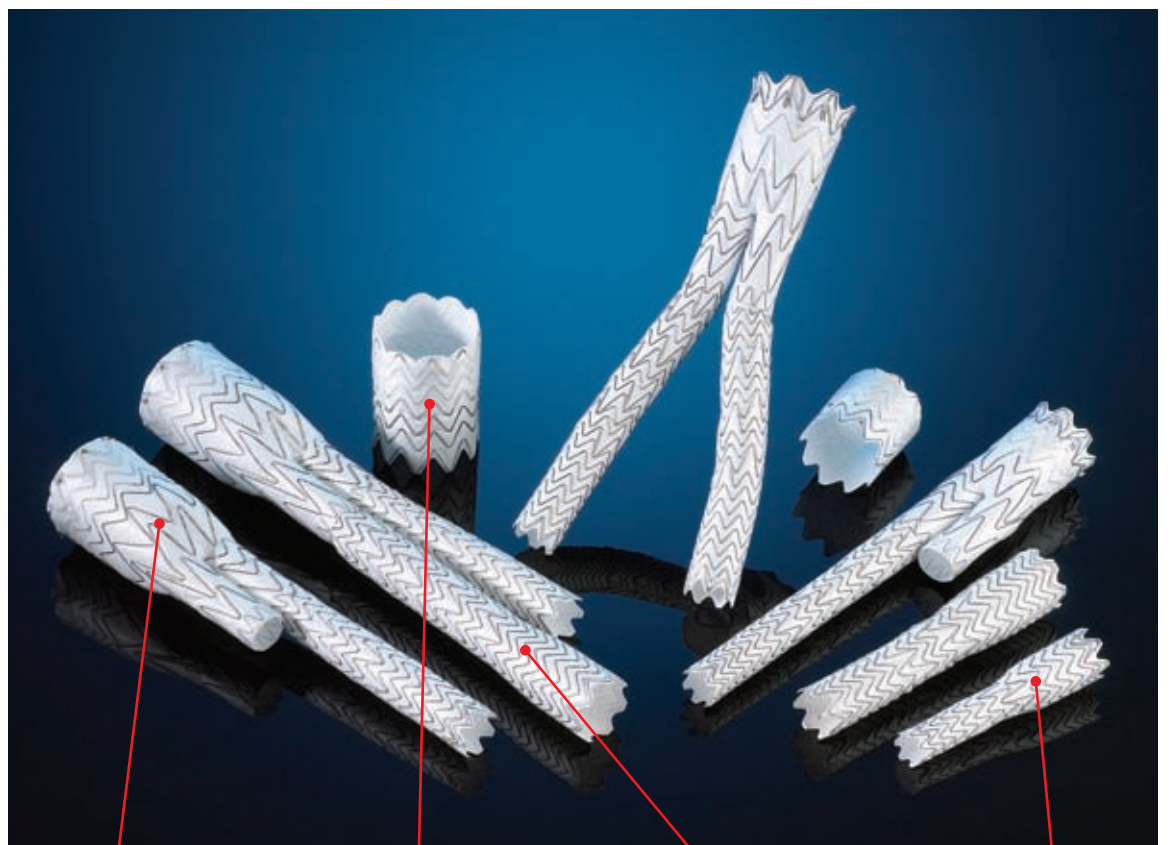

● Trunk-Ipsilateral ● Aortic Extender ● Contralateral Leg ● Iliac Extender