INSTRUCTIONS FOR USE FOR:

**Tru-Incise™
Valvulotome with
Interchangeable Heads
&
Tru-Incise
Valvulotome Catheter Kit**

en

ENGLISH

fr

FRANÇAIS

de

DEUTSCH

es

ESPAÑOL

it

ITALIANO

nl

NEDERLANDS

se

SVENSK

INSTRUCTIONS FOR USE FOR:

- **Tru-Incise Valvulotome with Interchangeable Heads**
- **Tru-Incise Valvulotome Catheter Kit**

en

NOTE:

The Tru-Incise Valvulotome Catheter Kit (REF # TIVC) is designed to be used in tandem with separately provided Tru-Incise cutting heads. In this case, these instructions for use pertain to use of the catheter kit in combination with separately packaged, individual cutting heads.

STERILITY:

This product is supplied sterile unless the integrity of the package has been compromised.

INDICATIONS:

In an in situ bypass procedure, the valves must be rendered incompetent to allow retrograde flow of arterial blood in the vein.

The Tru-Incise Valvulotome is specifically indicated for use during operations requiring rendering valve leaflets incompetent while minimizing intimal damage. This is primarily intended for use during in situ bypass procedures and may be utilized during distal infrainguinal bypass when a non-anatomic position is required (ex: profunda to anterior tibial artery), composite vein infrainguinal bypass, or even an aorto-renal bypass.

CONTRAINDICATIONS:

This device is not designed to be used for endarterectomy, vein stripping, embolectomy, vessel dilation, or thrombolysis.

TO OPEN THE PACKAGE:

Grasping the outer pouch, peel the pouch open exposing the tray. Using sterile technique, carefully remove the tray while placing it in a sterile field. Open the tray by peeling back the tray lid at the printed arrow, exposing the contents. Carefully remove the catheter and the remaining contents as needed.


POSSIBLE COMPLICATIONS WITH THE USE OF THE VALVULOTOME:

Complications which may occur with the use of any valvulotome, include, but are not limited to: **intimal disruption; vessel wall perforation; missed/incomplete valve disruption; phlebitis; thrombosis; stricture; A-V fistulae; entrapment of cutting heads; infection.**

INTRODUCTION:

Preparation of the saphenous vein for an in situ arterial bypass graft requires the disruption of the valve sets with minimal damage to the intima of the vein. The Tru-Incise Valvulotome is designed with this goal in mind.

Features of the Tru-Incise Valvulotome include the irrigation port at the end of the cutter, interchangeable cutting heads designed to optimize the cutting surface relative to a tapering vein, and the pronged blades that engage the valve sets for cutting, thus maximizing cutting action.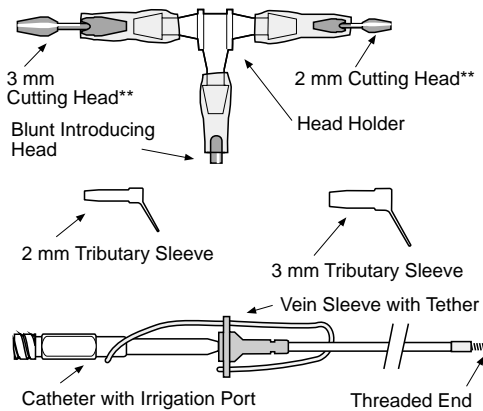

** Cutting heads are not included in the Tru-Incise Valvulotome Catheter Kit (REF # TIVC).


Valve disruption can be performed prior to or following anastomosis.

**SUGGESTED PROCEDURE - A:
VALVE DISRUPTION PRIOR TO ANASTOMOSIS**


This procedure relies on the use of delivered irrigant to insure closure of venous valve sets prior to disruption.

It should be noted that these instructions are not limited to femoral-to-below-knee bypasses. The techniques of manipulation of the valvulotome for rendering valves incompetent are equally applicable to other bypasses requiring valve disruption.

1. Expose and prepare the saphenous vein using standard technique. If the Tru-Incise Valvulotome is not going to be used to disrupt the first valve set, cut the first valve set under direct vision using standard technique.
2. Using the head holder, attach the blunt introducing head to the end of the catheter. Fully tighten the head prior to use (approximately eight full rotations).
3. Introduce the catheter with the blunt introducing head through the below knee end of the prepared vein. If desired, the vein sleeve can be affixed to the cut end of the vein using the attached tether to assist in easy passage of the catheter through the open end of the vein. Irrigate with heparinized saline (or other suitable solution) through the catheter irrigation port to dilate the vein prior to advancing the device into the vein.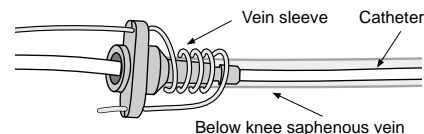

4. Gently clamp the end of the vein severed at the fossa ovalis from the deep vein to allow for vein dilation. With a 20 cc syringe, use gentle irrigation to minimize hydrostatic pressure injuries to the vessel wall. With the vein distended, gently advance the catheter while allowing it to float up the main channel of the saphenous vein.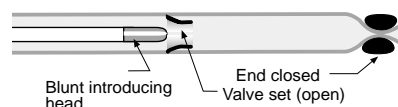


5. When the catheter reaches the clamped sapheno-femoral


vein end, release the clamp, advance the blunt introducing head beyond the end of the cut vein, and replace the blunt introducing head with an appropriately sized valve cutter head. To minimize the risk of cutting through a glove and to insure proper torque, the head holder should be used to attach the cutting head. Fully tighten the cutting head prior to use (approximately eight full rotations). **DO NOT ROTATE THE CATHETER DURING USE.** Rotation could cause unscrewing of the cutting head and/or damage to the vessel wall.

NOTE: Judgement of the surgeon best determines the appropriately sized cutting head which will best cut the valve sets at a given position in the vessel with the least damage to the vessel wall.

- Position the valve cutter at the sapheno-femoral end of the vein while injecting fluid through the catheter. This will distend the lumen and close the next valve set which will now be appropriately positioned for cutting.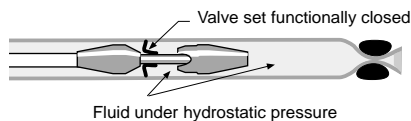

- Pull the valve cutter back to engage and cut the first or next valve set. Insure that the section of vein leading up to that valve set is fully distended to close the valves. Use slow and consistent traction to disrupt valve sets. While maintaining the hydrostatic pressure, pull the valve cutter to engage and cut each sequential valve set until all valves have been rendered incompetent within the appropriate section of vein relative to the chosen cutting head size.
- Change cutting heads as needed during the procedure. Cutting heads may be exchanged through the sapheno-femoral end of the vein or through a major tributary using the appropriate tributary sleeve. Appropriately match the size of the cutting head to the vessel's tapering lumen. The clearance between the vein and cutter head should be approximately 1/2 mm.

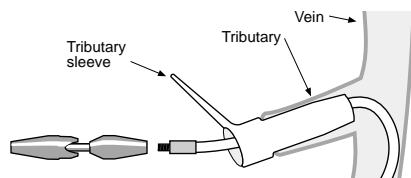- Withdraw the valve cutter until it engages the vein sleeve and remove the engaged cutter/sleeve assembly.
- Ligate the venous tributaries.
- When all valve sets are rendered incompetent, confirm free flow of irrigant through the saphenous vein. Complete the below knee anastomosis to the appropriate arterial segment. Confirm valve disruption and tributary ligation by doppler ultrasound, fluoro-angiography, angioscopy and/or duplex doppler.
- Interrupt any significant remaining A-V fistulae to insure adequate bypass flow.

SUGGESTED PROCEDURE - B:


VALVE DISRUPTION FOLLOWING ANASTOMOSIS

This procedure relies on systemic blood pressure to insure venous valve set closure prior to disruption.


It should be noted that these instructions are not limited to

femoral-to-below-knee bypasses. The techniques of manipulation of the valvulotome for rendering valves incompetent are equally applicable to other bypasses requiring valve disruption.

- Expose and prepare the saphenous vein using standard technique. If the Tru-Incise Valvulotome is not going to be used to disrupt the first valve set, cut the first valve set under direct vision using standard technique.
- Using the head holder, attach the blunt introducing head to the end of the catheter. Fully tighten the head prior to use (approximately eight full rotations).
- Introduce the catheter with the blunt introducing head through the below knee end of the prepared vein. If desired, the vein sleeve can be fixed to the cut end of the vein using the attached tether to assist in easy passage of the catheter through the open end of the vein. Irrigate with heparinized saline (or other suitable solution) through the catheter irrigation port to dilate the vein prior to advancing the device through the vein.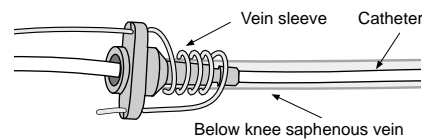


- Gently clamp the end of the vein severed at the fossa ovalis from the deep vein to allow for vein dilation. With a 20 cc syringe, use gentle irrigation to minimize hydrostatic pressure injuries to the vessel wall. With the vein distended, gently advance the catheter while allowing it to float up the main channel of the saphenous vein.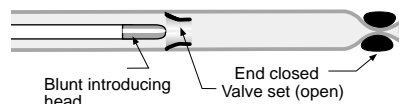


- When the catheter reaches the clamped sapheno-femoral vein end, release the clamp, advance the blunt introducing head beyond the end of the cut vein, and replace the blunt introducing head with an appropriately sized valve cutter head. To minimize the risk of cutting through a glove and to insure proper torque, the head holder should be used to attach the cutting head. Fully tighten the cutting head prior to use (approximately eight full rotations). **DO NOT ROTATE THE CATHETER DURING USE.** Rotation could cause unscrewing of the cutting head and/or damage to the vessel wall.

NOTE: Judgement of the surgeon best determines the appropriately sized cutting head which will best cut the valve sets at a given position in the vessel with the least damage to the vessel wall.

- Perform an anastomosis of the open sapheno-femoral vein end to the common femoral or other appropriate artery. Blood flow through the anastomosis will be retained at the first valve encountered.
- Withdraw the catheter to render the first or next valve set incompetent. Use slow and consistent traction to disrupt valve sets. Valve disruption will be confirmed as blood flow will now be retained at the site of the next restraining valve.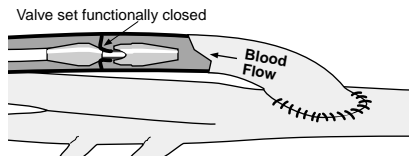

8. Change cutting heads as needed during the procedure. Cutting heads may be exchanged through a major tributary using the appropriate tributary sleeve or through a partially completed sapheno-femoral anastomosis. Appropriately match the size of the cutting head to the vessel's tapering lumen. The clearance between the vein and cutter head should be approximately 1/2 mm.

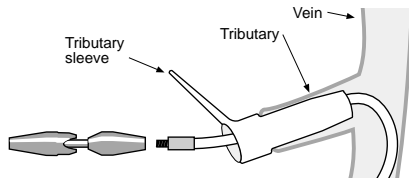9. Disrupt each valve set in a like manner.
10. Withdraw the valve cutter until it engages the vein sleeve and remove the engaged cutter/sleeve assembly.
11. Ligate the venous tributaries.
12. When all valve sets are rendered incompetent, confirm free flow of irrigant through the saphenous vein. Complete the below knee anastomosis to the appropriate arterial segment. Confirm valve disruption and tributary ligation by doppler ultrasound, fluoro-angiography, angiography and/or duplex doppler.
13. Interrupt any significant remaining A-V fistulae to insure adequate bypass flow.

WARNINGS/PRECAUTIONS:

1. Using an inappropriately sized cutting head may result in damage to the vein wall.
2. Veins are known to change diameter significantly in some individuals. Determine and consider the patient's anatomy when selecting valve cutter size.
3. During valve disruption, gently withdraw the catheter.
4. Do not over-distend the vessel with irrigation.
5. Fully tighten the cutting head prior to use. **DO NOT ROTATE THE CATHETER DURING USE.** Rotation could cause unscrewing of the cutting head and/or damage to the vessel wall.
6. Use the head holder to minimize the risk of cutting through a glove.
7. When using more than one cutter head to lyse valves, initiate the procedure with the largest head. This will aid in minimizing inadequate incising of valve leaflets.
8. When removing heads from the head holder, always grasp the catheter in one hand and the CENTER of the rigid portion of the head holder in the other and pull straight out.

REF

Item Number

Contents

QTY

Quantity

LOT

Manuf. Lot

Expiration Date

See Instructions for Use.

For Single Use only.

STERILE EO

Contents sterile unless package has been opened or damaged. Sterilized by ethylene oxide.

MFG. BY

Manufactured by

DIST. BY

Distributed by

EU REP

Authorized European Representative

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©2002 UreSil, L.P.
UreSil is a registered trademark of UreSil, L.P.
Tru-Incise is a trademark of UreSil, L.P.
**Manufactured in U.S.A. by UreSil, L.P., Skokie, IL.
To Reorder: Call 1-800-538-7374 (U.S.A. only) or fax